



Sun Diagnostics, LLC.  
 60 Pineland Drive Brunswick Hall, Suite 322  
 New Gloucester, ME 04260  
 Toll free: 1-877-786-3424 [www.sundiagnosics.us](http://www.sundiagnosics.us)

**ASSURANCE™ Interference Test Kit** is for Laboratory Use as part of Interference testing procedures. Interferents available include: CREATININE, GLUCOSE, GLYCEROL LACTATE, PYRUVATE, UREA, URIC ACID and 0.1N NaOH as the base for URIC ACID. Components may be purchased individually, in customer-defined combinations, or as INT-06 (one vial of each component).

**KIT COMPONENTS AND TYPICAL VALUES**

Interferent	Typical Concentration	Reorder Number
CREATININE	8840 µmol/L	INT-06C
GLUCOSE	1100 mmol/L	INT-06GLU
GLYCEROL	13.92 mmol/L	INT-06GLY
LACTATE	132 mmol/L	INT-06LAC
PYRUVATE	6180 µmol/L	INT-06PYR
UREA	858 mmol/L	INT-06U
URIC ACID	28 mmol/L	INT-06UA
NaOH <i>(for UA control pool)</i>	0.1N	
<b>ASSURANCE™ Interference Test Kit – 1 Vial of each Component</b>		<b>INT-06</b>

**REAGENTS**

This product is prepared from purified chemicals. Human source materials are not used.

**NONREACTIVE INGREDIENTS**

None

**PRECAUTIONS AND WARNINGS**

Dispose of all waste material in accordance with requirements of your local waste management authorities.

**STORAGE AND STABILITY**

This product will be stable until the expiration date when stored at -10 to -20°C. After use, refreeze promptly. A maximum of 4 Freeze/Thaw cycles is recommended.

**LIMITATIONS**

- Typical concentrations are intended as a guide only. **Laboratory should determine actual values before proceeding with interference experiment(s).**
- INT-06 should not be used past the expiration date listed on the kit box. Refer to Interferent vials for expiration of individually purchased components or custom kit configurations.
- If there is evidence of microbial contamination discard the vial.
- This product is not intended for use as a control, standard or calibrator.
- Information gathered from experiments performed using ASSURANCE™ Interference Test Kit should not be used to adjust patient results. Confirm results with an alternate method when necessary.

Sample Interference Testing procedure (based on CLSI EP-7A guidelines), dilution calculator and results reporting spreadsheet are available at [www.sundiagnosics.us](http://www.sundiagnosics.us) or email [support@sundiagnosics.us](mailto:support@sundiagnosics.us).

Symbols Used			
	Biohazard		Lot Number
	Protect from light		Consult Package Insert
	Manufacturer		Reorder Number
	Expiration	ASSURANCE™ Interference Test Kit is a general purpose reagent intended for laboratory use. CE marking not required	
	Storage Temp		