



Technical Bulletin #25-004

Date	September 22, 2025
Part Number(s)	INT-01T
Product Name(s)	ASSURANCE™ Triglyceride-Rich Lipoprotein Interferent
Product Lot Number and Expiration Date	Lot: 0119025 Exp: 2026-10-31
Subject	Recommendation to use positive displacement pipette with this lot of material.

Dear Valued Customer,

Thank you for using our INT-01T, Triglyceride-Rich Lipoprotein product for your Lipemia interference testing. The above listed lot (*0119025, Exp. 2026-10-31*) is more highly concentrated than previous lots, making it more viscous which could lead to difficulties when using an air-displacement pipette. We recommend using a **positive displacement pipette for this material**. These pipettes are ideal because they use a capillary piston within the tip, which directly contacts the liquid and eliminates the air cushion found in air-displacement pipettes. This direct contact prevents the fluid's properties (like viscosity) from affecting the accuracy of the volume dispensed.

Please retain this bulletin for your records and contact Sun Diagnostics, LLC with any questions.

Regards,

Technical Support
Sun Diagnostics, LLC
Email: support@sundiagnosics.us
Phone: 1-877-786-3424

TB25-004 TRL 0119025